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All-in-One solution with hyaluronic acid for cleaning, disinfecting, storing, rinsing and hydrating all soft and color contact lenses. Prevents proteins. The sodium hyaluronate polymer keeps the contact lenses hydrated and provides a pleasant wearing comfort.

IMPORTANT DIRECTIONS FOR USE AND SAFETY INFORMATION, PLEASE READ CAREFULLY AND KEEP FOR FUTURE REFERENCE. IMPROPER CONTACT LENS CARE CAN LEAD TO SERIOUS DAMAGE TO THE EYES

DIRECTIONS FOR USE

Before handling the contact lenses, wash your hands thoroughly, rinse completely, and dry them with a clean towel. Open the lens case. Remove lens and place in the palm of your hand. Apply some drops of the solution to each lens surface. Gently rub the lens in circular motion for a few seconds. Rinse the lens thoroughly with the solution. Place the lens in the lens case and fill it with solution. Allow the lenses to soak for at least 4 hours or over night. Before using the lenses, rinse them again with the solution. Discharge used solution from the lens case. Rinse the case with the solution and leave it dry. NEVER USE TAP WATER TO RINSE THE CASE AND HOLDER. If the contact lenses remain in the solution for more than 7 days, the cleaning process must be repeated prior to wearing them.

PRECAUTIONS AND WARNINGS

Check the dropper tip is sealed before first use. If already opened, do not use. ONCE BOTTLE IS OPEN, DISCARD ANY REMAINING SOLUTION AFTER 60 DAYS. Close bottle after use. Store between 5° and 30° F. Do not use the product after expiry date. Never touch the dropper tip of the bottle to any surface, since this may contaminate the solution. Replace lens case with every new bottle. Consult your eye care professional before changing solutions. Always use fresh solution. Never re-use the solution a second time. Never transfer the solution to another container.

Never dilute or mix with other solutions. DO NOT use tap water to rinse the contact lenses; only use a saline solution or multipurpose solution.

Consult your eye care professional before changing solutions. Do not use in case of hypersensitivity or allergy to any component. If any reactions such as inflammation, persistent itching or irritation or other eye problems occur, remove the contact lenses and stop using the product. Contact your eye care professional or ophthalmologist. Before taking any medication, consult your eye care professional or ophthalmologist. KEEP OUT OF REACH OF CHILDREN. DO NOT INGEST.

CONTRAINDICATIONS

No contraindications reported.

INGREDIENTS

Ethylene diamine tetra acetic acid 0.01%, PolyHexaMethyleneBiguanide 0.00015%, Hydroxyethylcellulose, Macromolecular Hyaluronic Acid, Sodium chloride, Dibasic sodium phosphate, Monobasic sodium phosphate, Anhydrous Sodium Hydroxide, Purified water.

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